

Summary of Prospective Clinical Study Featuring American Diagnostica – ACTICLOT® LA Complete™ Combined dRVVT and dilute PT tests

Study Center: Duke University Medical Center, Durham, NC USA –
Dr. Jogin Wu, Director

Patient Cohort: 529 consecutive citrated plasma samples sent to Dept. of Pathology for laboratory determination of lupus anticoagulants (LA)

Testing Protocol

Each sample was tested using Duke's standard LA panel :


- 1) LS-aPTT/ hexagonal phase (Diagnostica Stago StaClot-LA) to confirm positive
- 2) dRVVT : DVVtest/DVVconfirm (American Diagnostica Inc.)

Each sample was also tested with ACTICLOT LA Complete (dPT + DVV)

Results

Samples were scored positive when **both** screen and confirm procedures were positive for any test.

Data Chart

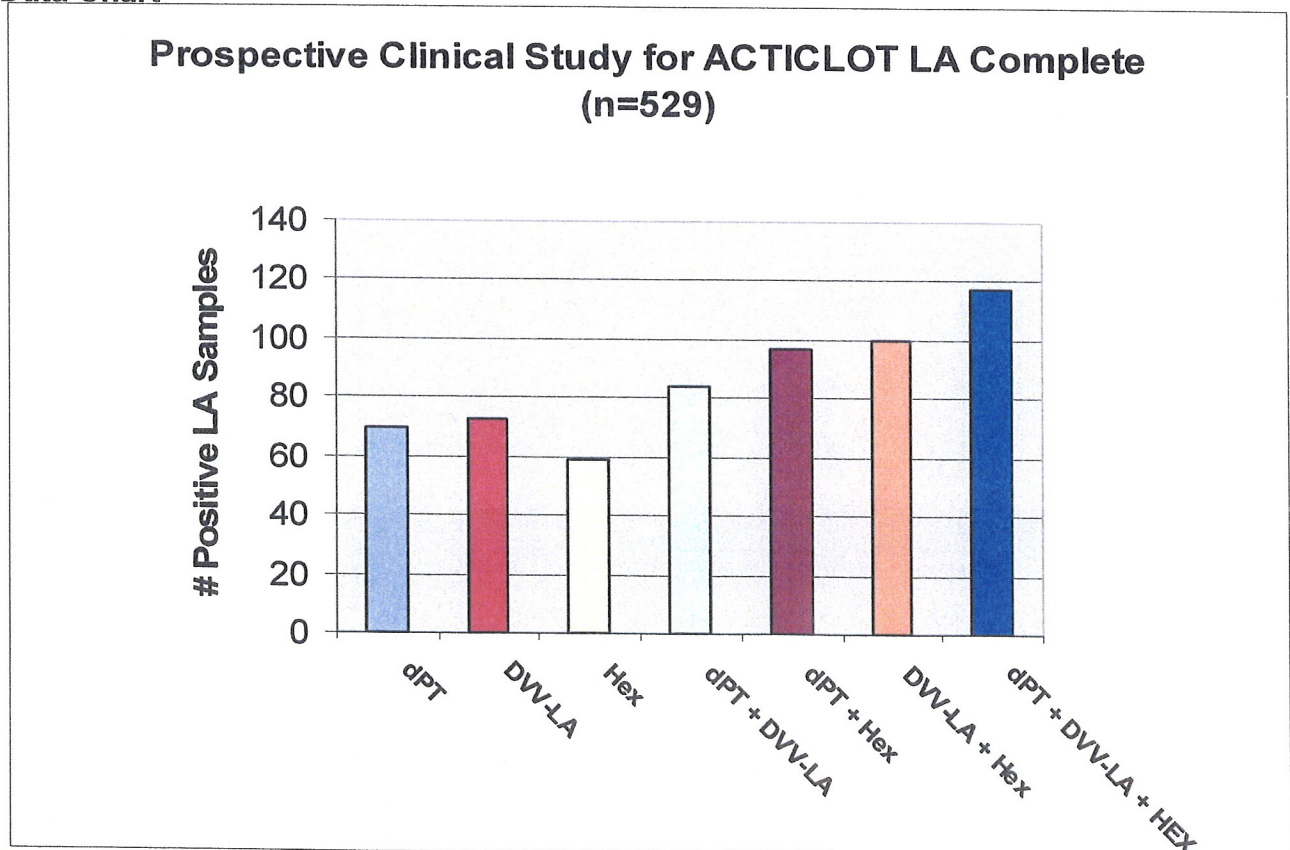

Conclusions: ACTICLOT LA Complete found 17% more LA patients than when using only the dRVVT + hexagonal phase tests.

American Diagnostica Inc., Stamford, CT USA Tel: 1.203.602.7777, Fax: 1.203.602.2221

Email: LINUS@amdiag.com

www.americandiagnostica.com