



ANALYSING A PATIENT SAMPLE

PREPARE FOR TESTING

Before starting up – please note!

- The Test Cartridge must reach an operating temperature of 18-30°C before use.
- Be sure that the Analyzer is switched on and ready for use.
- The Test Cartridge has to be analysed within 1 minute from capillary filling.



Consult the Afinion™ HbA1c Package Insert for further information on sample material and handling.



1

Open the foil pouch just before use. Use the handle for correct grip.

COLLECT SAMPLE

2




Pull the sampling device straight up from the Test Cartridge.

3



Fill the capillary completely with patient sample. Do not wipe off the capillary.

4

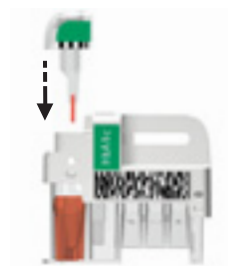


Immediately replace the sampling device into the Test Cartridge.

START TESTING

5



Touch  to enter patient sample mode. The lid opens automatically.

6



Insert the Test Cartridge with the barcode label facing left.

7





Close the lid manually.

RECORD RESULT


8



Touch  and enter patient ID during processing. Touch  to confirm.

9



Record the result, then touch  to accept. The lid opens automatically.

10



Remove and discard the used Test Cartridge. Close the lid manually.

ANALYSING A CONTROL

PREPARE FOR TESTING

Before starting up – please note!

- The Test Cartridge and the control material must reach an operating temperature of 18-30°C before use.
- Be sure that the Analyzer is switched on and ready for use.
- Mix the control material thoroughly before use.
- The Test Cartridge has to be analysed within 1 minute from capillary filling.



Consult the Afinion™ HbA1c and HbA1c Control Package Inserts for further information on sample material and handling.




- 1 Open the foil pouch just before use. Use the handle for correct grip.

COLLECT SAMPLE



- 2 Pull the sampling device straight up from the Test Cartridge.

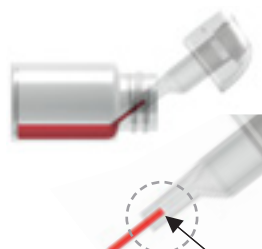

- 3 Fill the capillary completely with control material. Do not wipe off the capillary.



- 4 Immediately replace the sampling device into the Test Cartridge.

START TESTING



- 5 Touch to enter control mode. The lid opens automatically.



- 6 Insert the Test Cartridge with the barcode label facing left.



- 7 Close the lid manually.

RECORD RESULT



- 8 Touch and enter control ID during processing. Touch to confirm.



- 9 Record the result, then touch to accept. The lid opens automatically.



- 10 Remove and discard the used Test Cartridge. Close the lid manually.